



Recommended drilling sequence guide

for Interna BTI implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

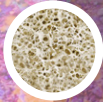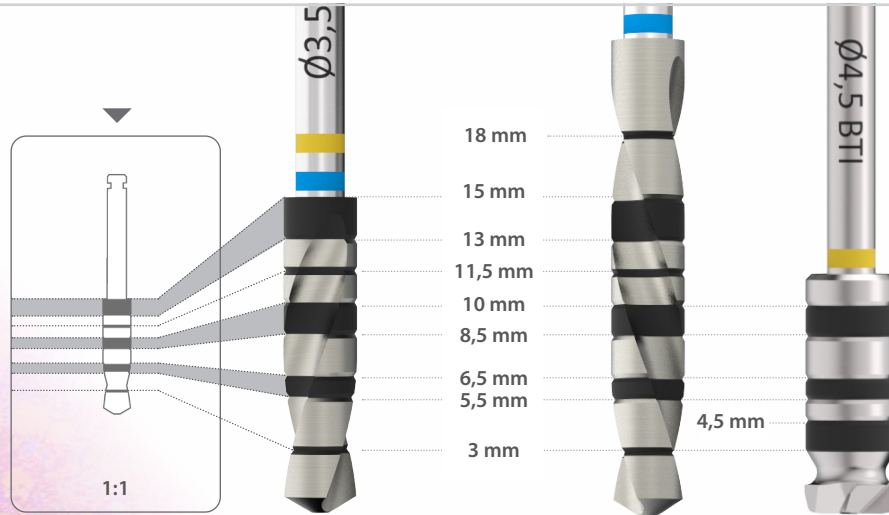

CLASS / HOUNSFIELD

0	1.400 - 1.200	III	800 - 500
I	1.200 - 1.000	IV	500 - 400
II	1.000 - 850	V	350 - 100

DRILL

Initial drill
Drill
Countersink drill
Front cutting drills

REVOLUTIONS

800 - 1.000 rpm
50 - 75 rpm
150 - 200 rpm
50 - 75 rpm


IRRIGATION

yes
no
no
no

Implant lengths (from 6,5 to 13 mm)




 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

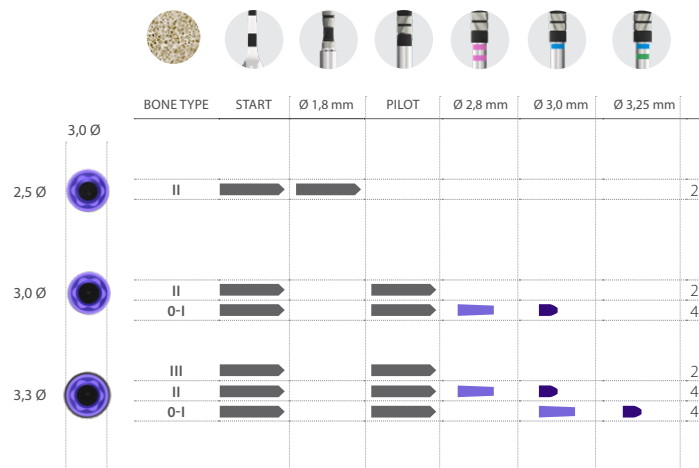

Implant lengths (from 4,5 to 5,5 mm)




 Total length of the implant
 2-3 mm drilling depth

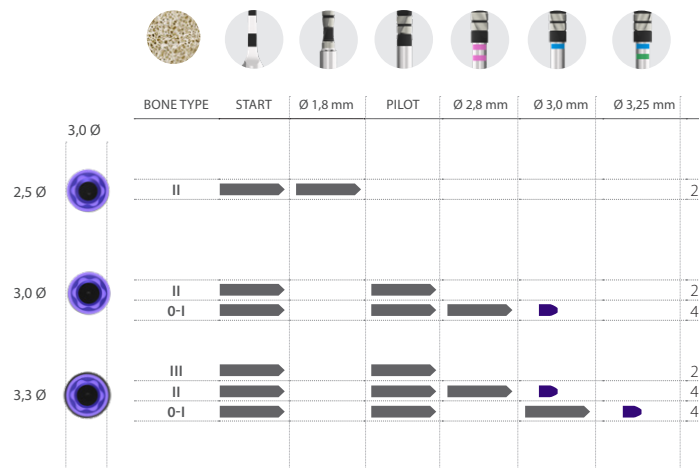

Implant lengths (from 7,5 to 15 mm)


 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

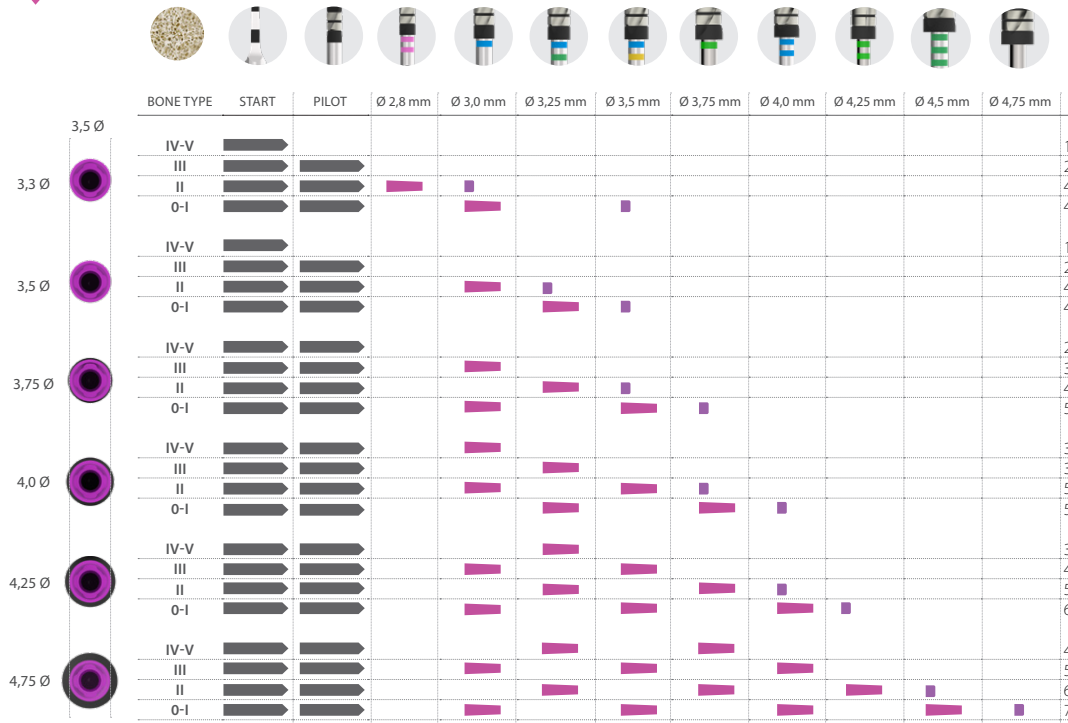



Implant lengths *(from 4,5 to 6,5 mm)*

— Total length of the implant
X.X Ø Total length of the implant with specified diameter drill
■ 2-3 mm drilling depth

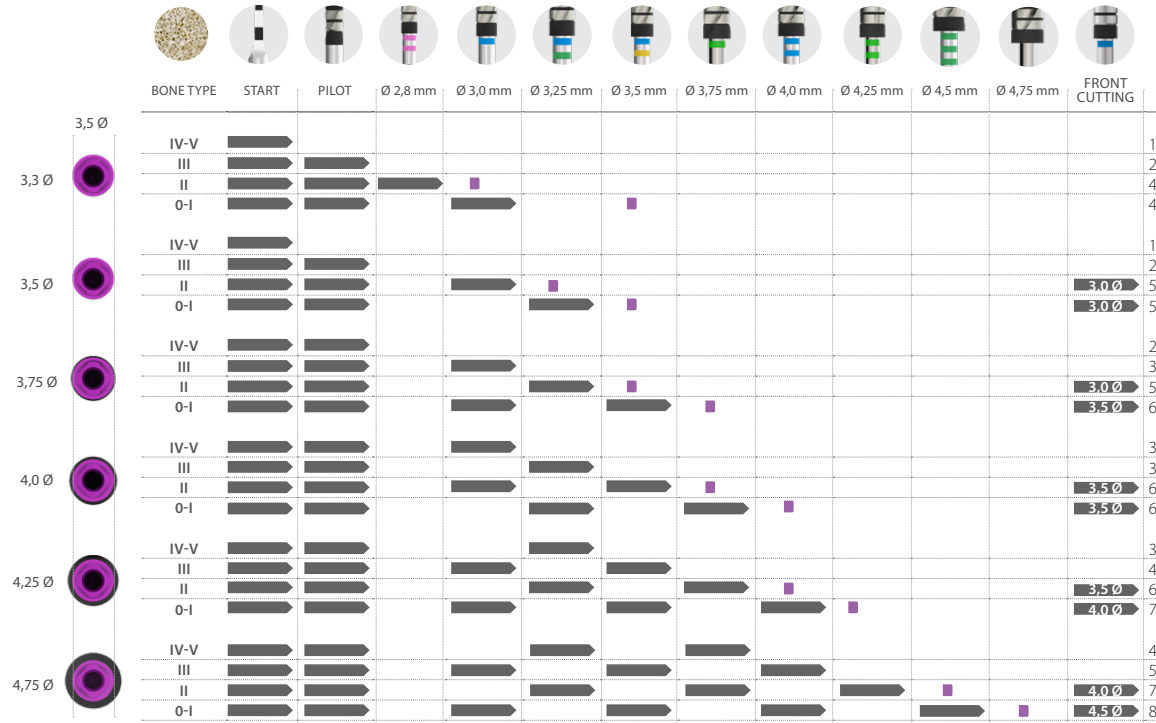


○ Narrow / CORE Platform
Platform diameter: Ø3,5 mm

drilling
sequence guide

bt.
Human
Technology

Implant lengths (from 6,5 to 13 mm)


 Total length of the implant
 5,5 mm drilling depth

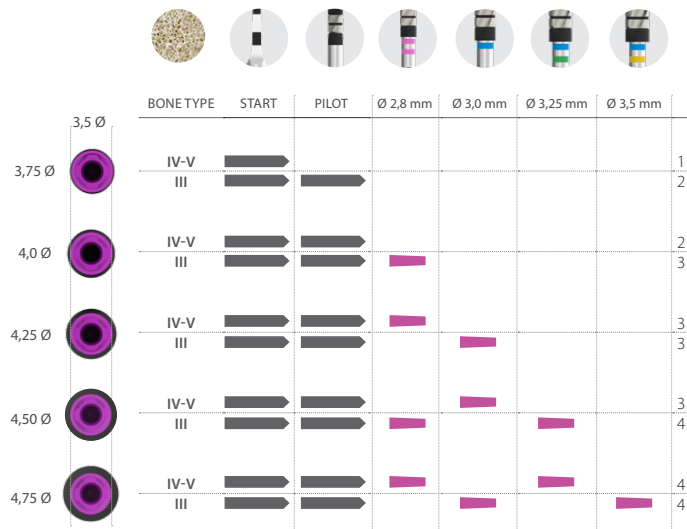

IMPORTANT:
Drilling sequence valid only
for CORE-X® implants

Implant lengths (from 4,5 to 5,5 mm)

➡ Total length of the implant


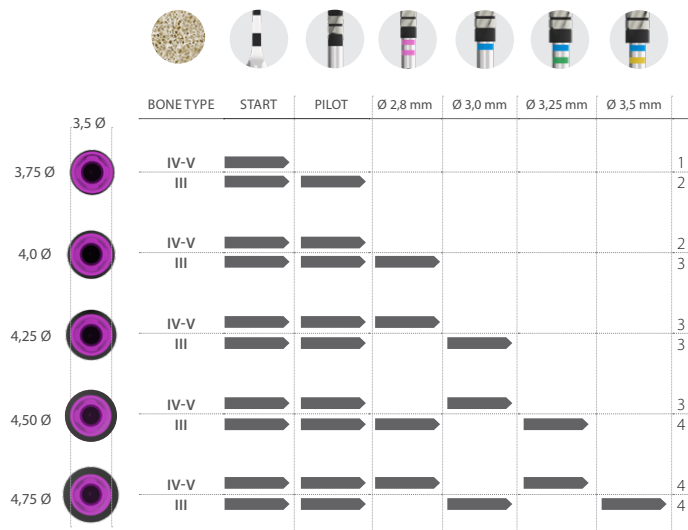


IMPORTANT:
Drilling sequence valid only
for CORE-X® implants



Implant lengths


(from 10 to 15 mm)

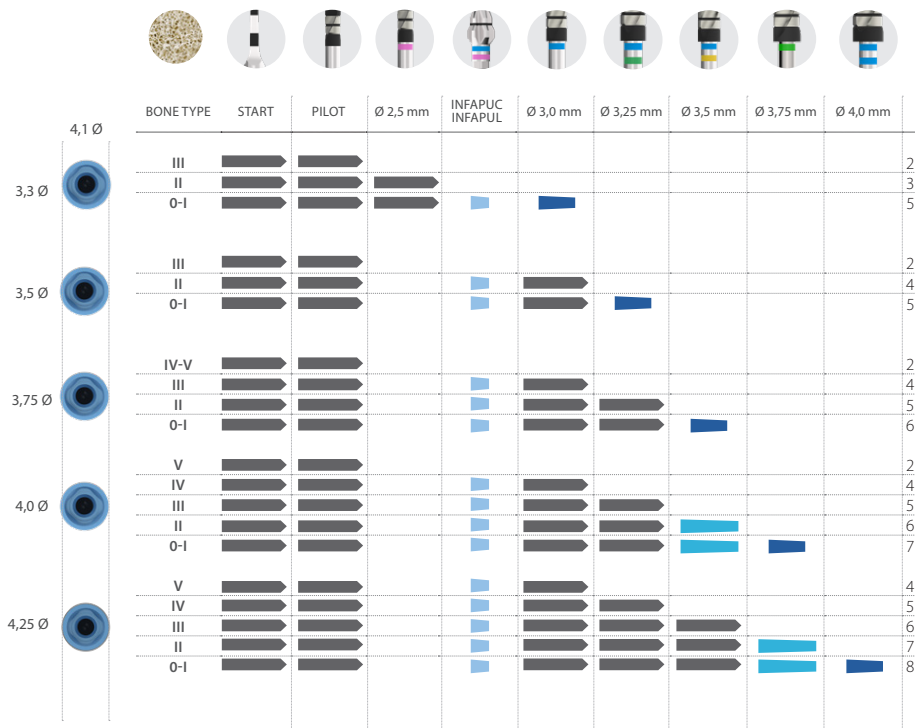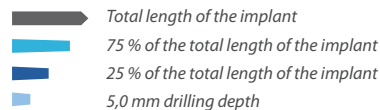

drilling
sequence guide

Implant lengths (from 5,5 to 8,5 mm)


- Total length of the implant
- X.X Ø Total length of the implant with specified diameter drill
- 5,0 mm drilling depth
- 2-3 mm drilling depth

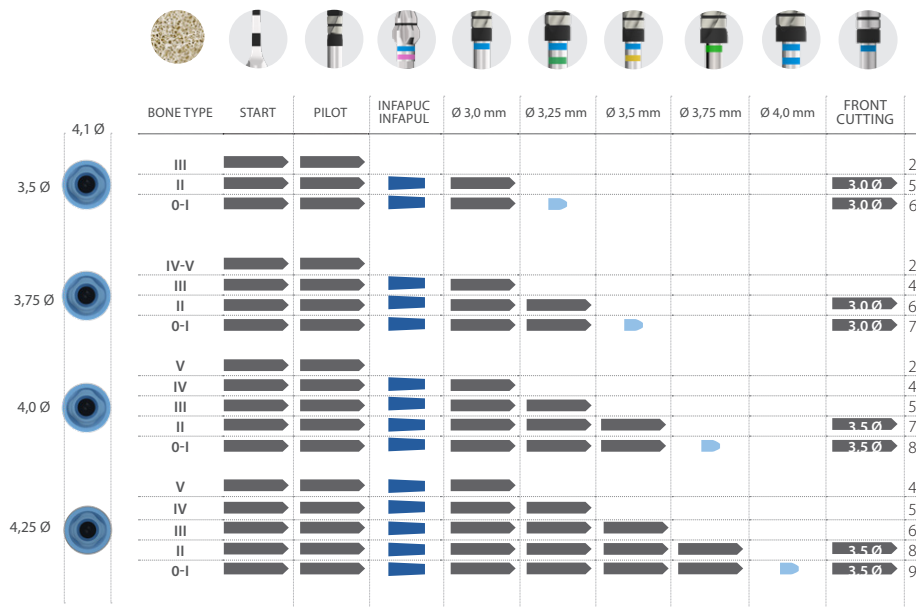

Implant lengths (from 8,5 to 15 mm)



 Total length of the implant
 6,5 mm drilling depth
 3 mm drilling depth

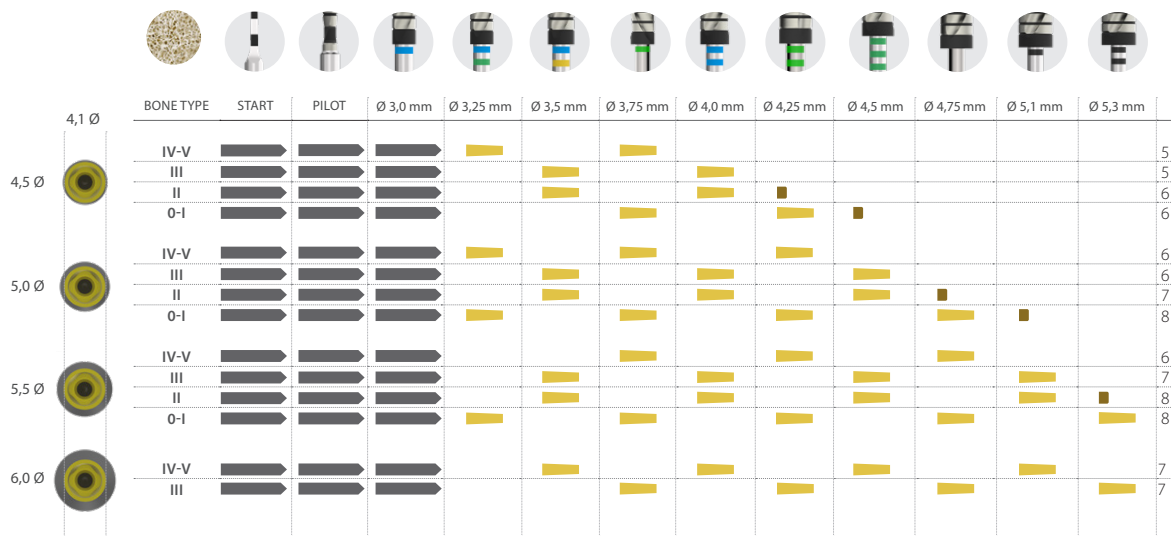




Implant lengths

(from 4,5 to 7,5 mm)

- Total length of the implant
- X,X Ø Total length of the implant with specified diameter drill
- 3 mm drilling depth

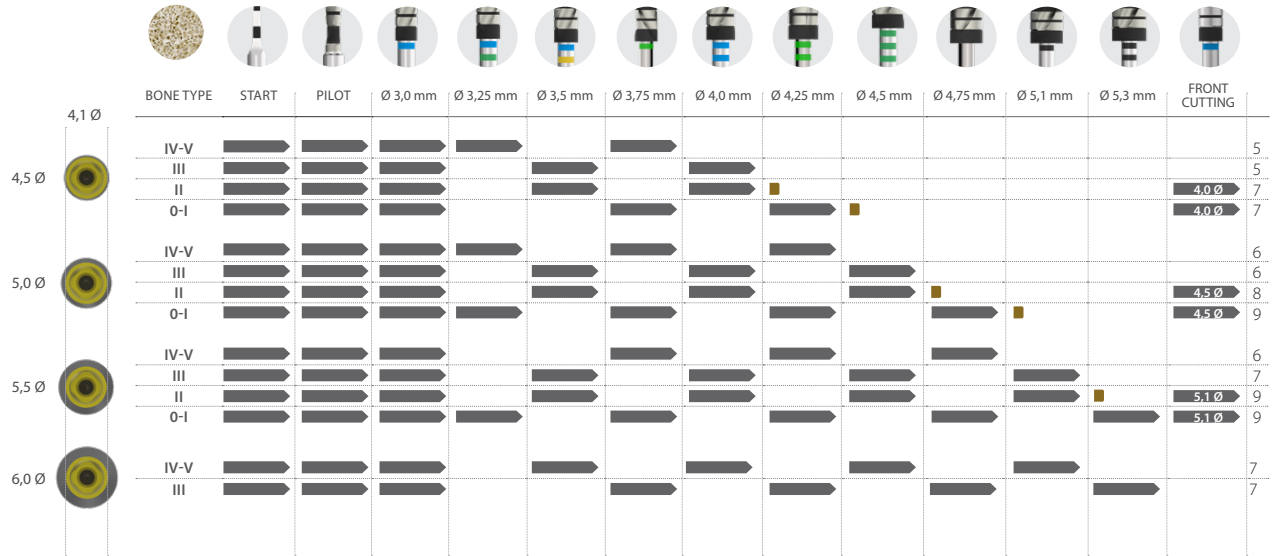

Universal Plus Platform
Platform diameter Ø4,1 mm

drilling
sequence guide

