



Recommended drilling sequence guide

for Interna BTI implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

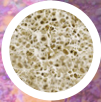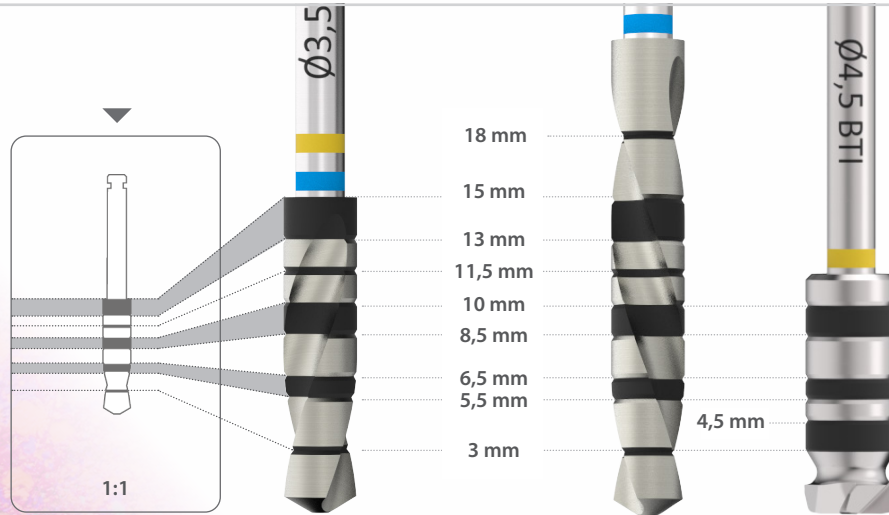

CLASS / HOUNSFIELD

0	1.400 - 1.200	III	800 - 500
I	1.200 - 1.000	IV	500 - 400
II	1.000 - 850	V	350 - 100

DRILL

Initial drill
Drill
Countersink drill
Front cutting drills

REVOLUTIONS

800 - 1.000 rpm
50 - 75 rpm
150 - 200 rpm
50 - 75 rpm


IRRIGATION

yes
no
no
no

Implant lengths

(from 6,5 to 13 mm)


 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

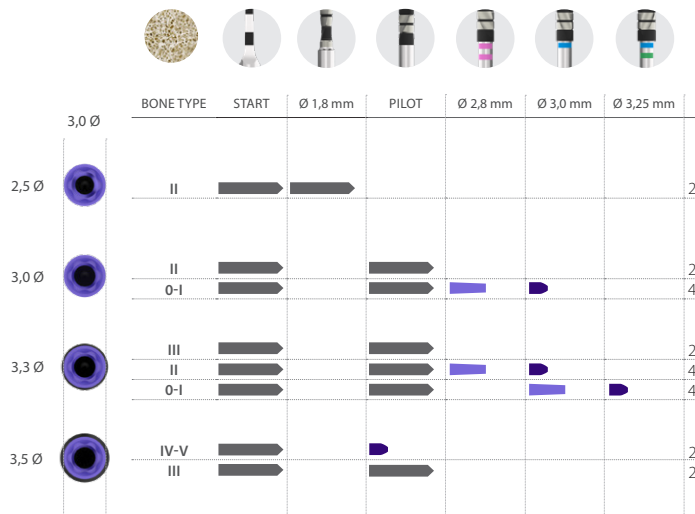

Implant lengths

(from 4,5 to 5,5 mm)

 Total length of the implant
 2-3 mm drilling depth

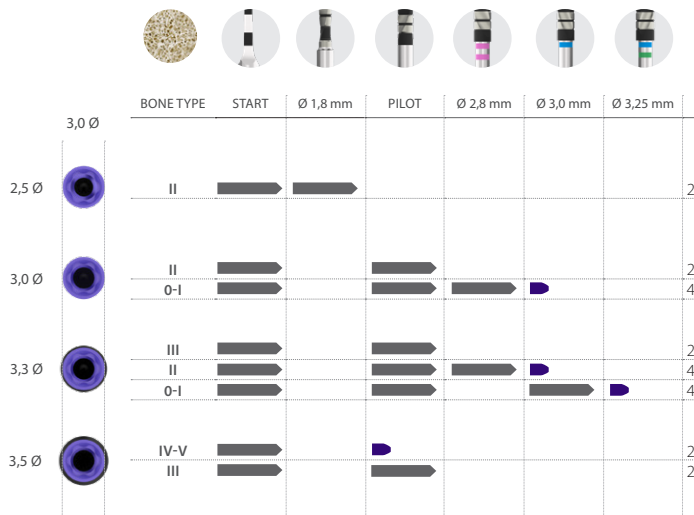

Narrow Platform CORE

Platform diameter Ø3,5 mm




Implant lengths

(from 7,5 to 15 mm)



Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

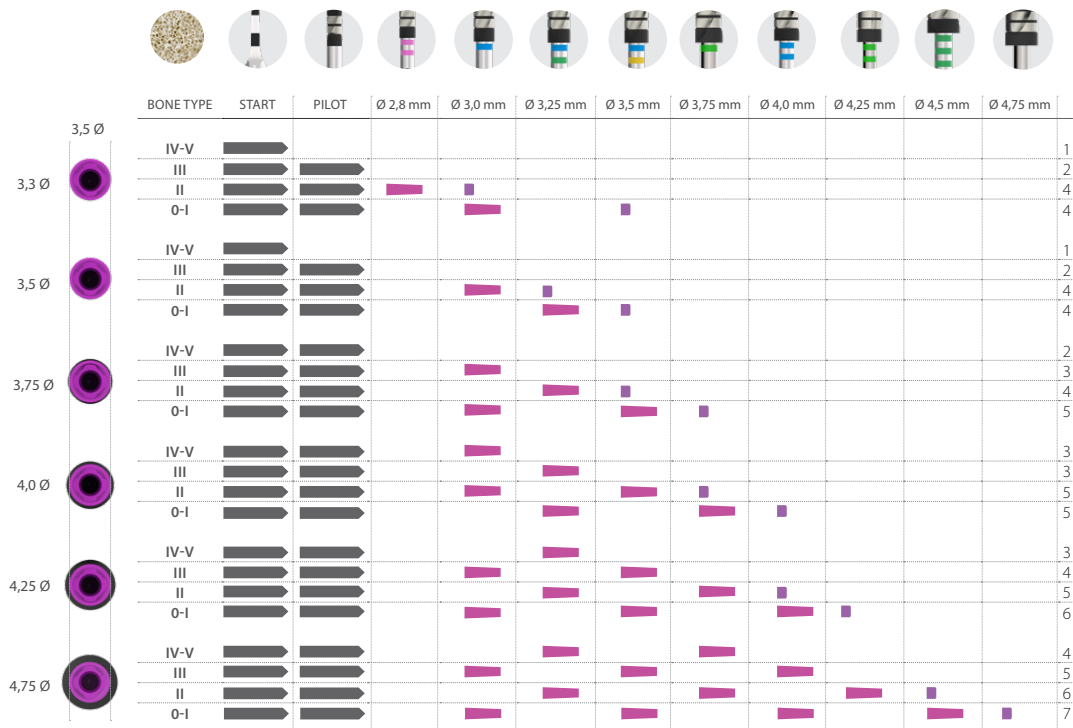


drilling
sequence guide



Implant lengths

(from 4,5 to 6,5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 2-3 mm drilling depth

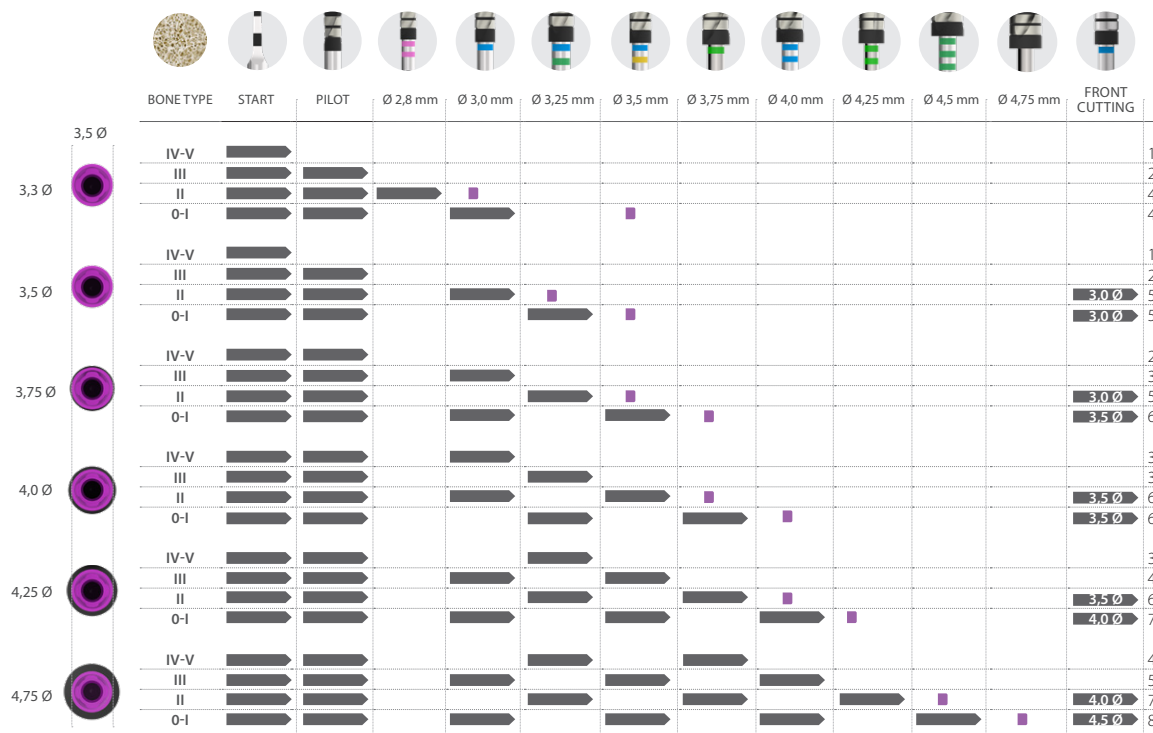


Narrow Platform CORE-X

Platform diameter Ø3.5 mm

Implant lengths

(from 6,5 to 13 mm)

 Total length of the implant
 5,5 mm drilling depth
 2-3 mm drilling depth

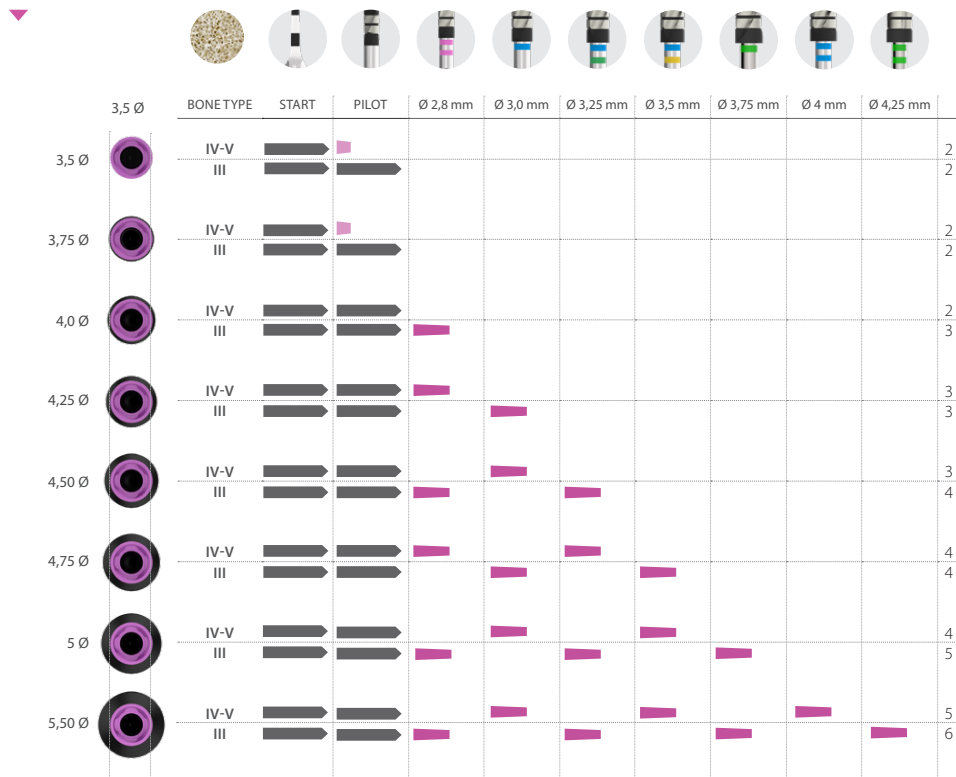

IMPORTANT:

Drilling sequence
valid only for CORE-X
implants

drilling
sequence guide




Narrow Platform CORE-X

Platform diameter Ø3,5 mm

Implant lengths

(from 4,5 to 5,5 mm)

 Total length of the implant
 2-3 mm drilling depth

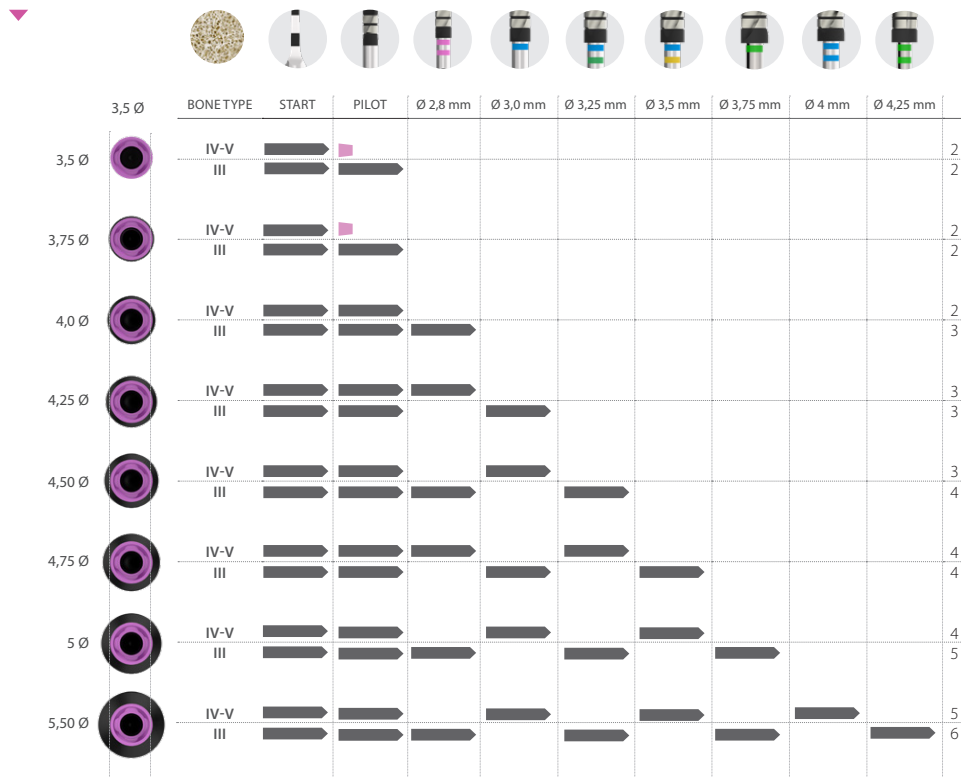



IMPORTANT:
 Drilling sequence
 valid only for CORE-X
 implants

drilling
sequence guide



Implant lengths


(from 10 to 15 mm)

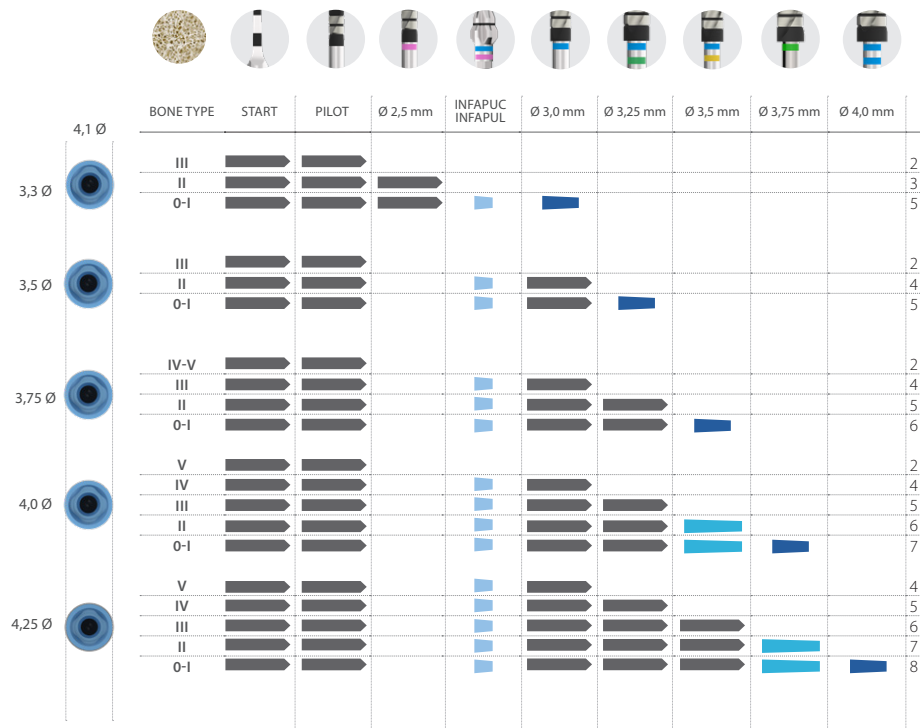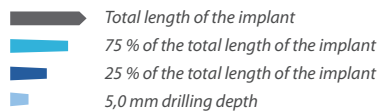

Implant lengths

(from 5,5 to 8,5 mm)


-  Total length of the implant
-  Total length of the implant with specified diameter drill
-  5,0 mm drilling depth
-  2-3 mm drilling depth

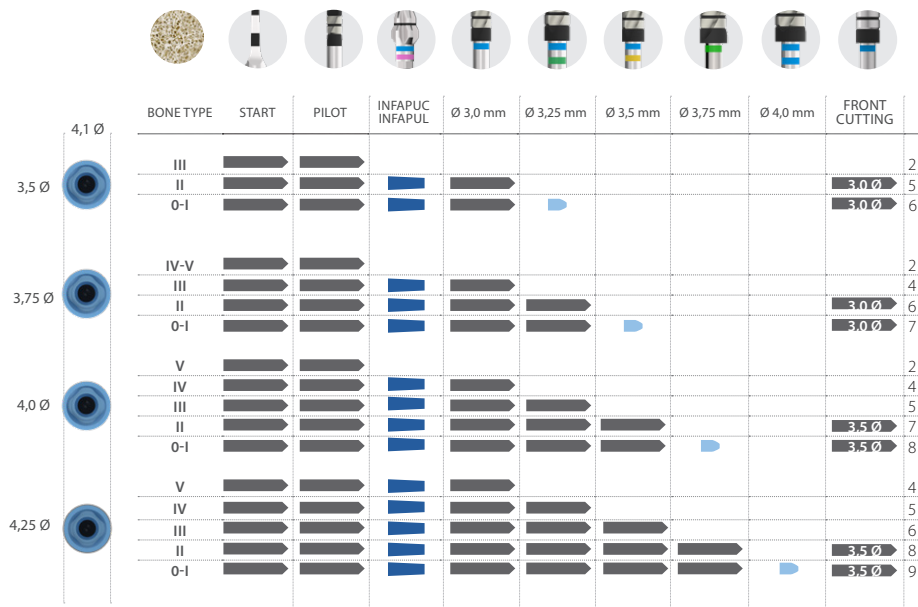

Implant lengths

(from 8,5 to 15 mm)

 Total length of the implant
 6,5 mm drilling depth
 3 mm drilling depth

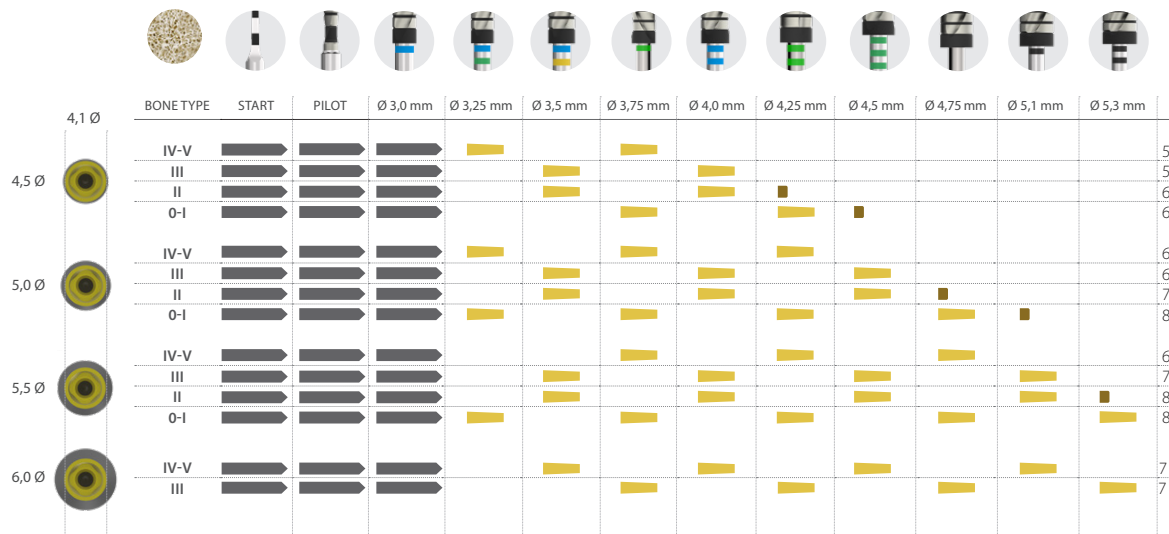

Implant lengths

(from 4,5 to 7,5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 3 mm drilling depth

