



Recommended drilling sequence guide

for Interna BTI implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

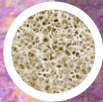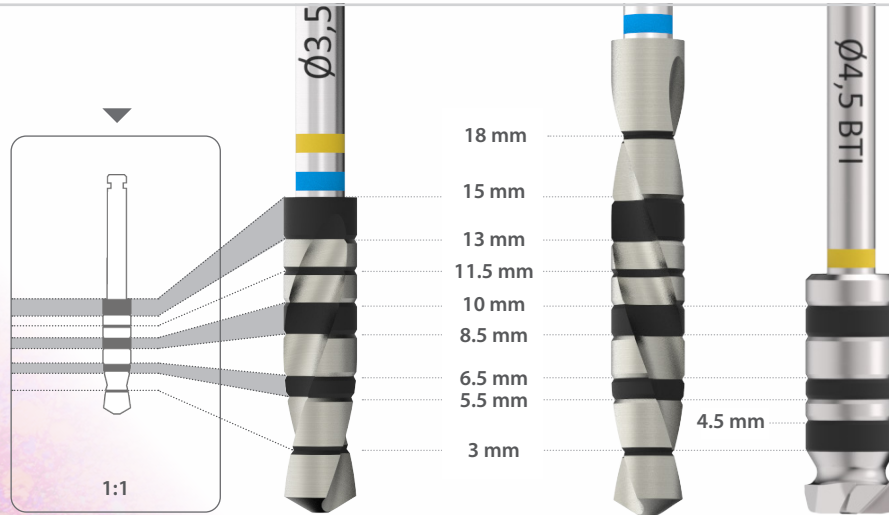

CLASS / HOUNSFIELD

0	1.400 - 1.200	III	800 - 500
I	1.200 - 1.000	IV	500 - 400
II	1.000 - 850	V	350 - 100

DRILL

Initial drill
Drill
Countersink drill
Front cutting drills

REVOLUTIONS

800 - 1.000 rpm
50 - 75 rpm
150 - 200 rpm
50 - 75 rpm

IRRIGATION

yes
no
no
no

Implant lengths

(from 5.5 to 13 mm)

 Total length of the implant
 mm Variable
 2-3 mm drilling depth


IMPLANT LENGTHS	2.5 Ø						
	BONE TYPE	START	Ø 1.8 mm	Ø 2.0 mm	Ø 2.2 mm	Ø 2.3 mm	Ø 2.4 mm
5.5 - 6.5 - 7.5 mm	III						2
	II			5.5 mm			3
	0-I			5.5 mm	5.5 mm	5.5 mm	6
8.5 - 10 mm	III						2
	II			6.5 mm			3
	0-I			6.5 mm	6.5 mm	6.5 mm	6
11.5 - 13 mm	III						2
	II			8.5 mm			3
	0-I			8.5 mm	8.5 mm	8.5 mm	6



Implant lengths

(from 7.5 to 13 mm)

 Total length of the implant
 6.5 mm drilling depth
 2-3 mm drilling depth

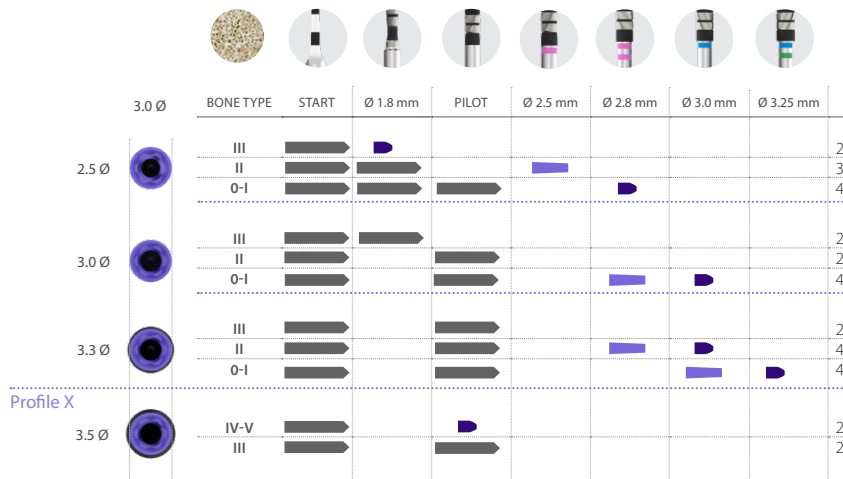


Implant lengths

(from 4.5 to 6.5 mm)




- Total length of the implant
- Total length of the implant with a drill of the specified diameter
- 2-3 mm drilling depth
- 2-3 mm drilling depth. Only for 6.5 mm implants

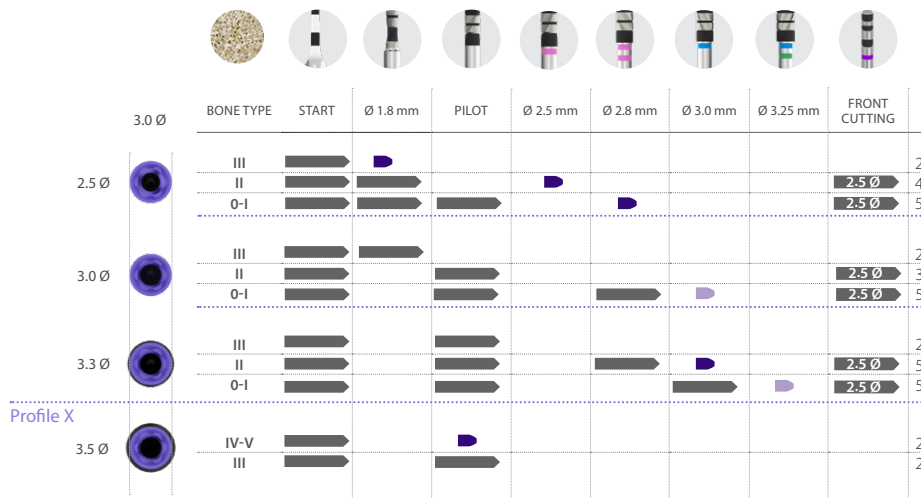

Implant lengths

(from 7.5 to 15 mm)


 Total length of the implant
 6.5 mm drilling depth
 2-3 mm drilling depth

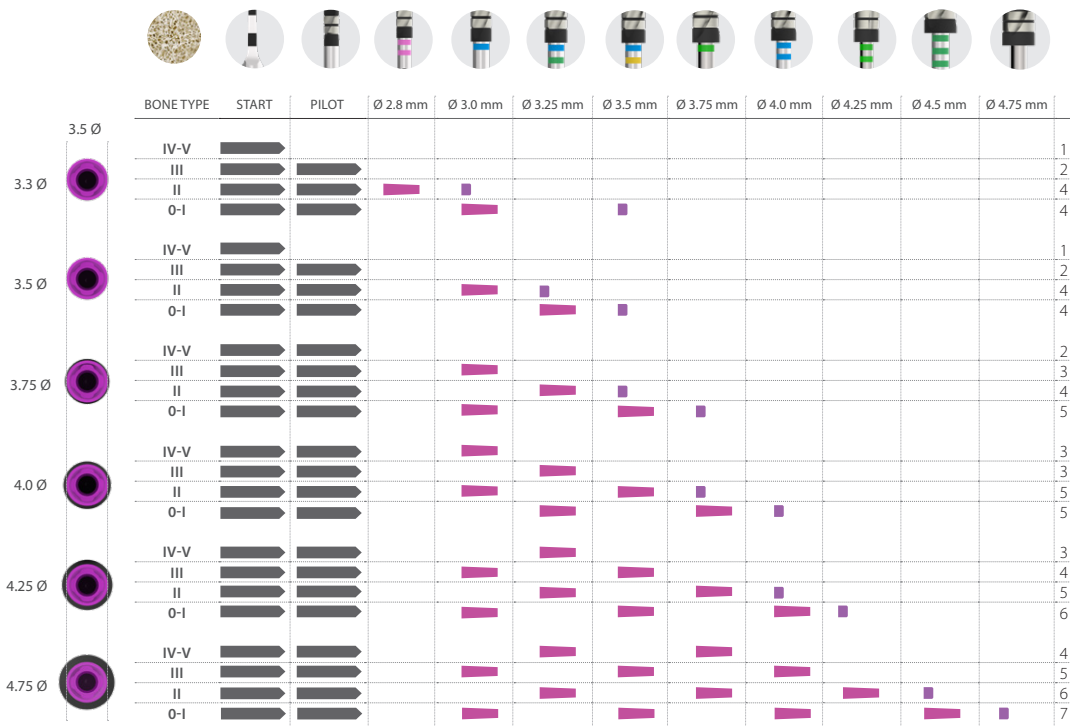

Implant lengths

(from 4.5 to 6.5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 2-3 mm drilling depth

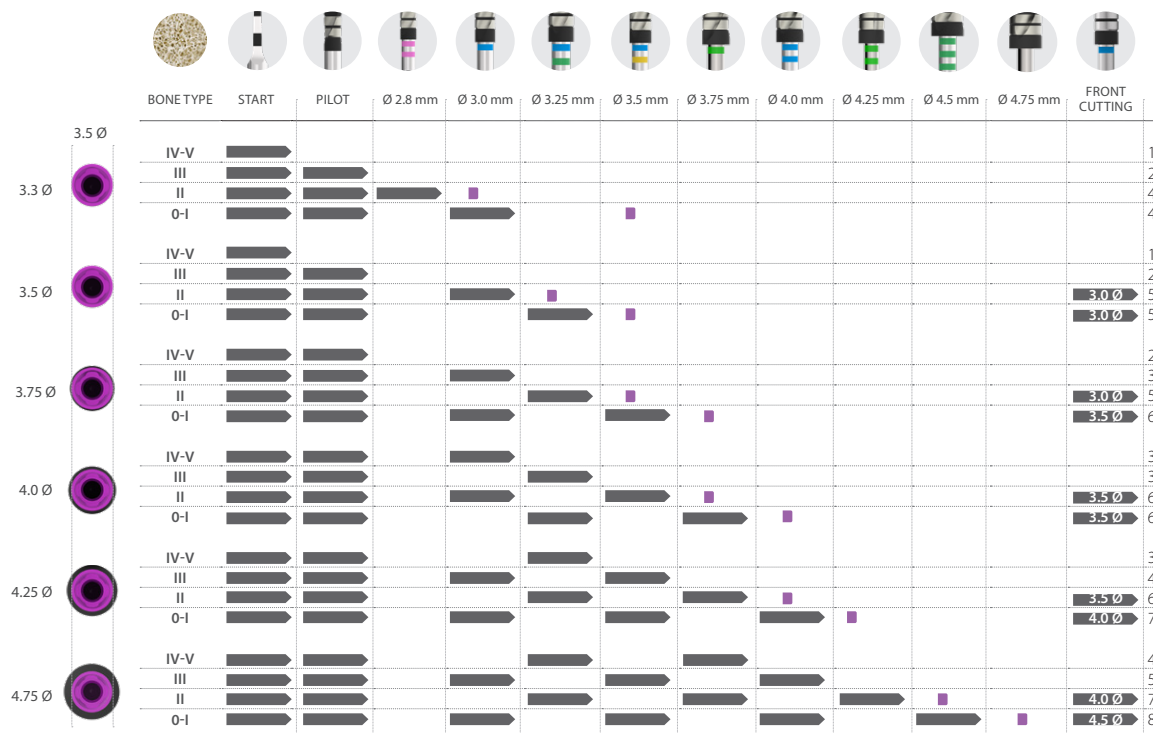

Narrow Platform CORE-X


Platform diameter Ø 3.5 mm



Implant lengths

(from 6.5 to 13 mm)

 Total length of the implant
 5.5 mm drilling depth
 2-3 mm drilling depth

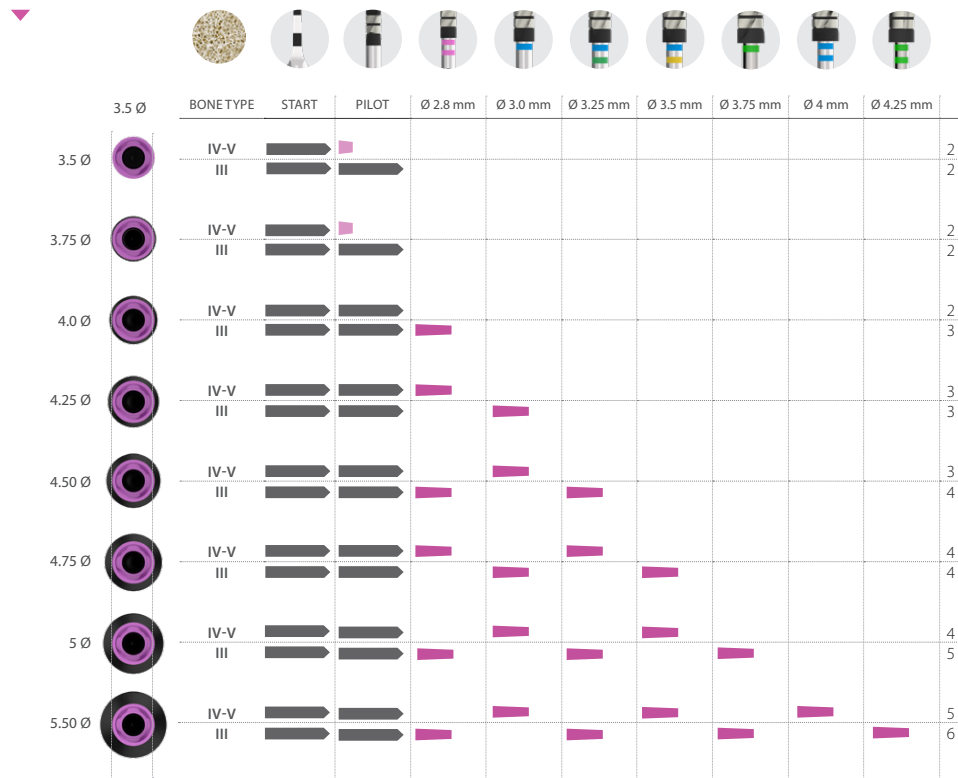

IMPORTANT:
Drilling sequence
valid only for CORE-X
implants

drilling
sequence guide




Narrow Platform CORE-X

Platform diameter Ø 3.5 mm

Implant lengths

(from 4.5 to 5.5 mm)

 Total length of the implant
 2-3 mm drilling depth

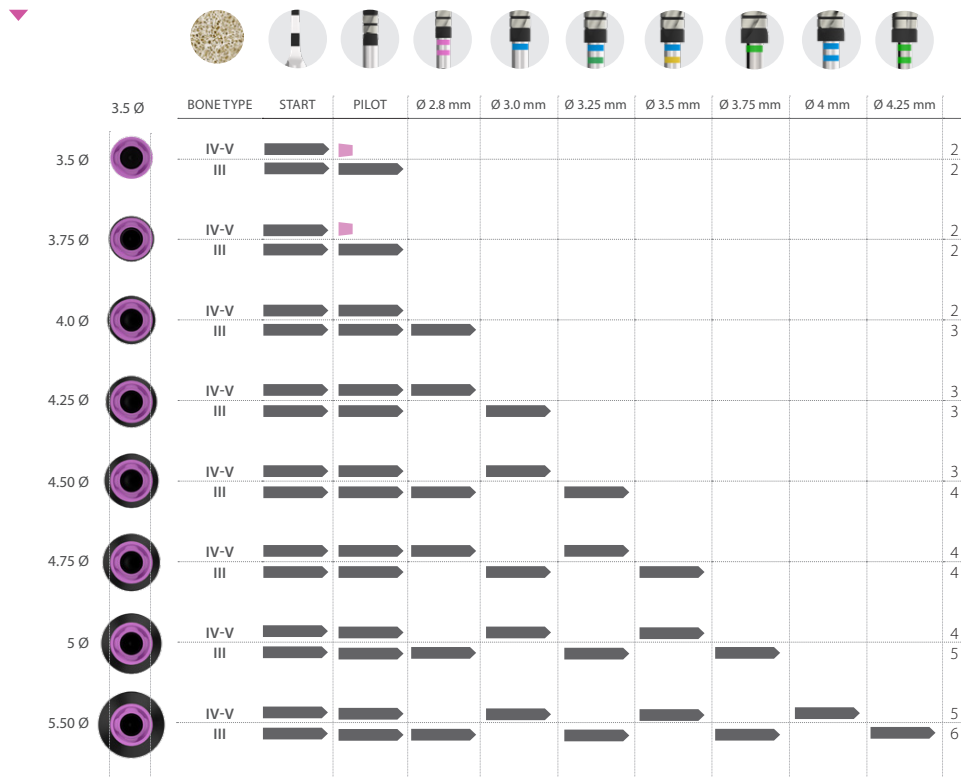



IMPORTANT:
Drilling sequence
valid only for CORE-X
implants

drilling
sequence guide



Implant lengths


(from 10 to 15 mm)

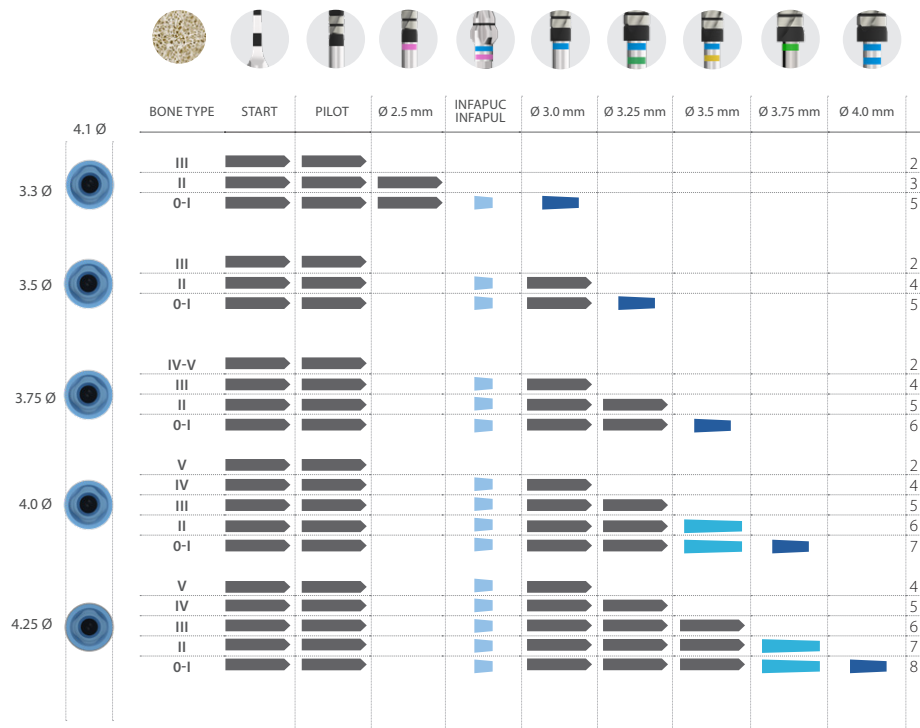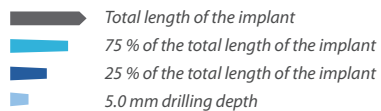

Implant lengths

(from 5.5 to 8.5 mm)


-  Total length of the implant
-  Total length of the implant with specified diameter drill
-  5.0 mm drilling depth
-  2-3 mm drilling depth

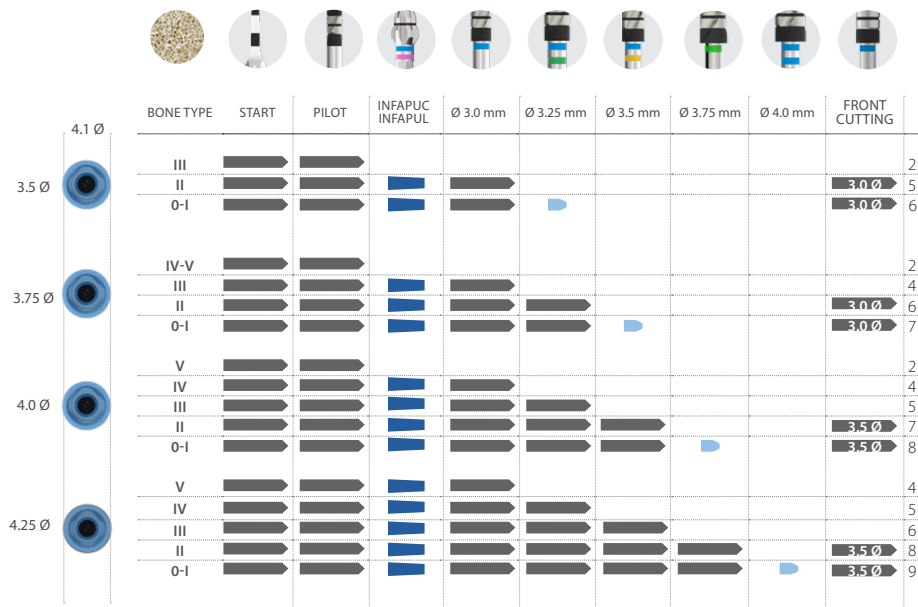

Implant lengths

(from 8.5 to 15 mm)

 Total length of the implant
 6.5 mm drilling depth
 3 mm drilling depth

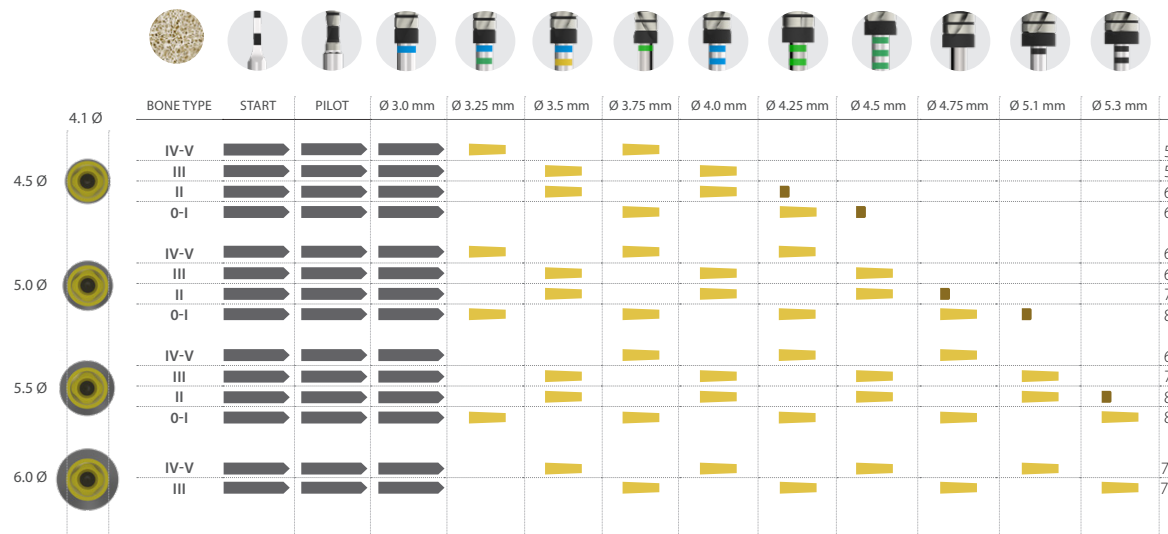

Implant lengths

(from 4.5 to 7.5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 3 mm drilling depth

