



Recommended drilling sequence guide

for Interna BTI implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

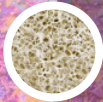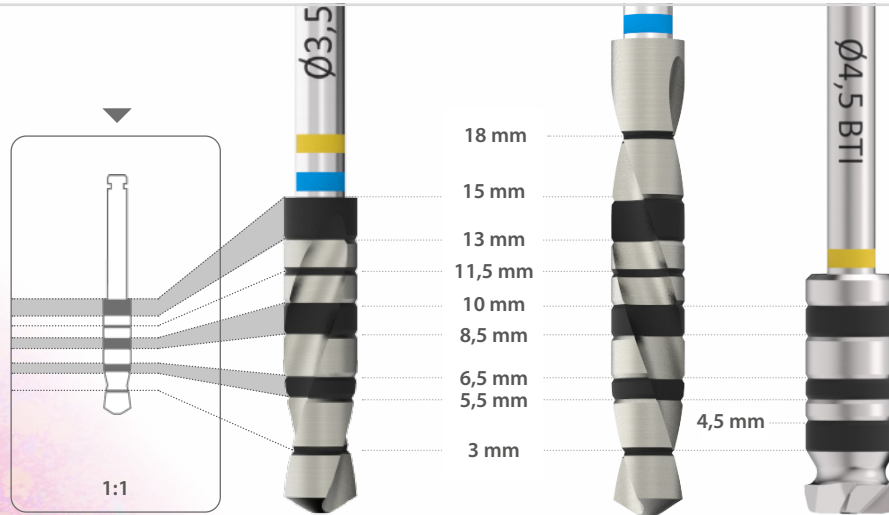

Class/Hounsfield

I 1050 - 1400

II 850 - 1000

III 550 - 800

IV 400 - 500

V 100 - 350

DRILL

Initial drill

Drill

Countersink drill

Front cutting drills

REVOLUTIONS

800 - 1.000 rpm

50 - 75 rpm

150 - 200 rpm

50 - 75 rpm

IRRIGATION

yes

no


no

no

Implant lengths (from 6,5 to 13 mm)



 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

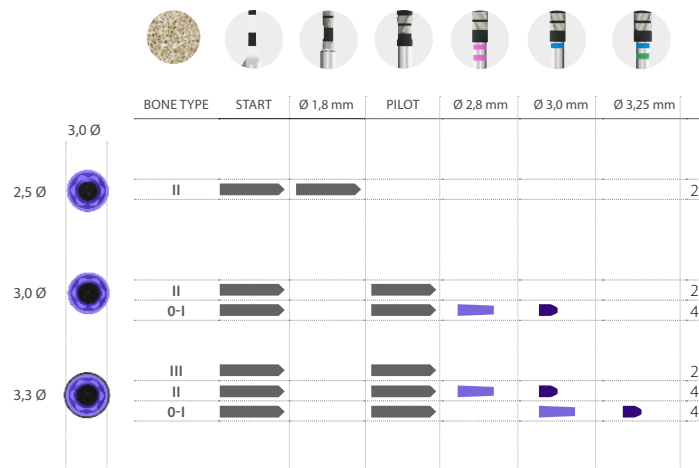


Implant lengths

(5,5 mm)




 Total length of the implant
 2-3 mm drilling depth

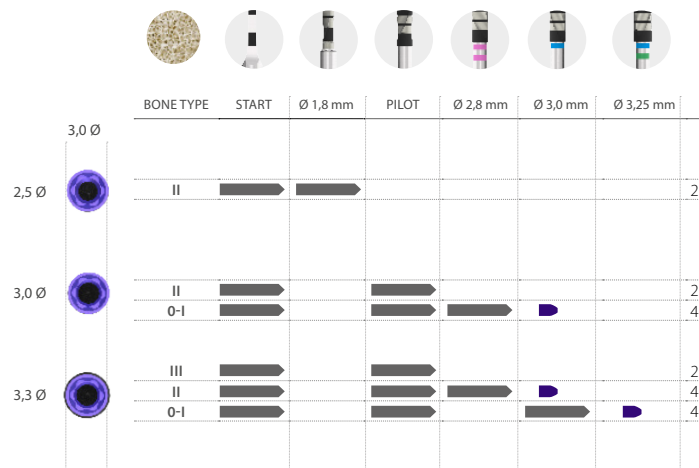

Implant lengths (from 7,5 to 15 mm)


 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth

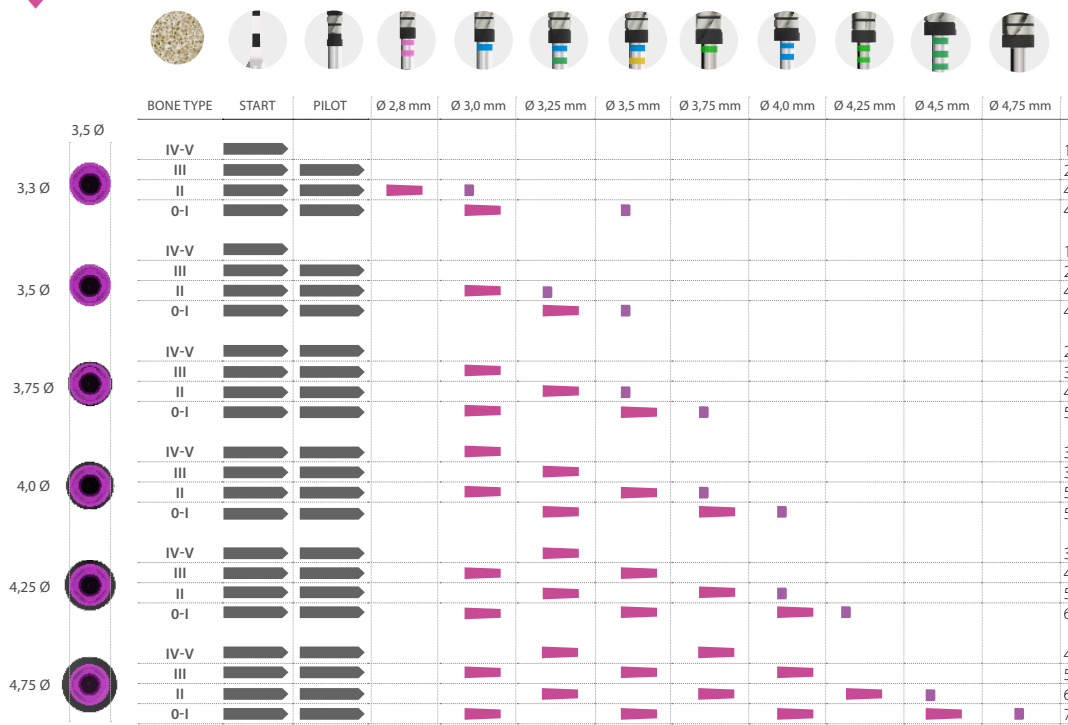




Implant lengths *(from 4,5 to 6,5 mm)*

— Total length of the implant
— X.X Ø Total length of the implant with specified diameter drill
■ 2-3 mm drilling depth

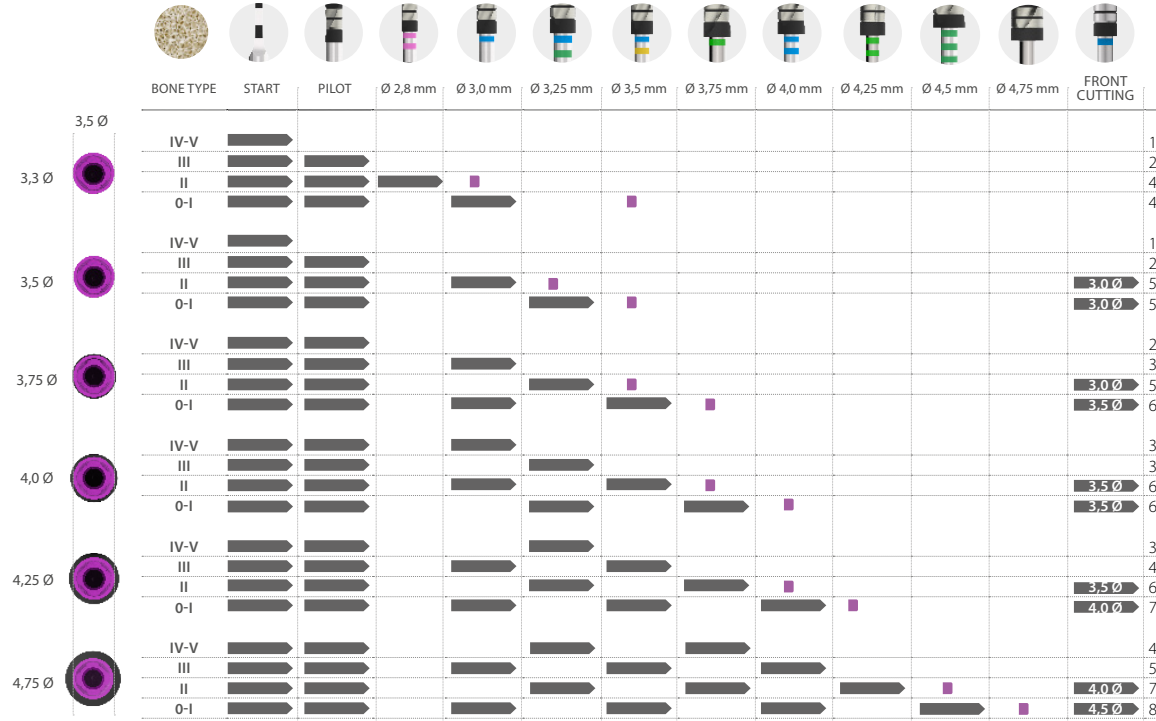

Implant lengths (from 6,5 to 13 mm)


 Total length of the implant
 5,5 mm drilling depth

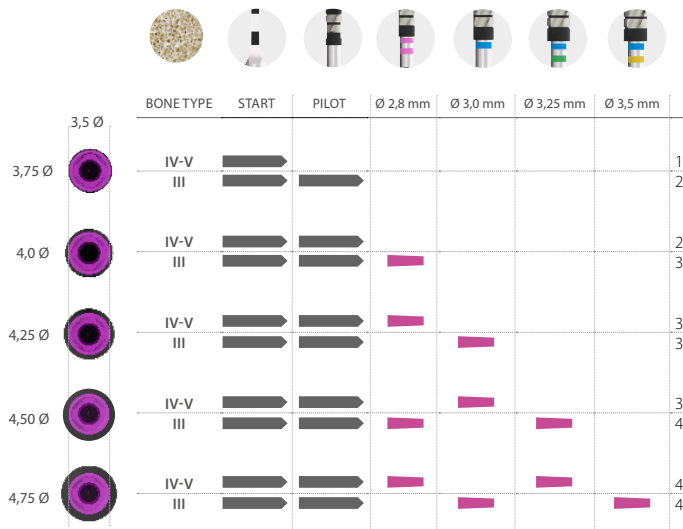

IMPORTANT:
Drilling sequence valid only
for CORE-X® implants

Implant lengths (from 4,5 to 5,5 mm)

➡ Total length of the implant

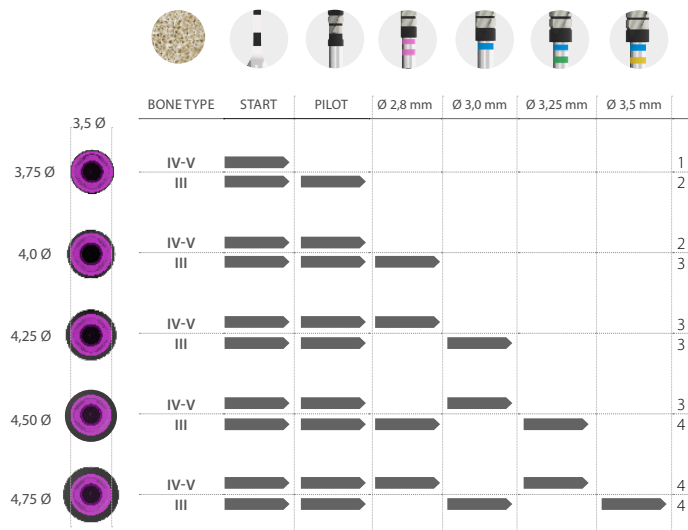



IMPORTANT:
Drilling sequence valid only
for CORE-X® implants





Implant lengths


(from 10 to 15 mm)

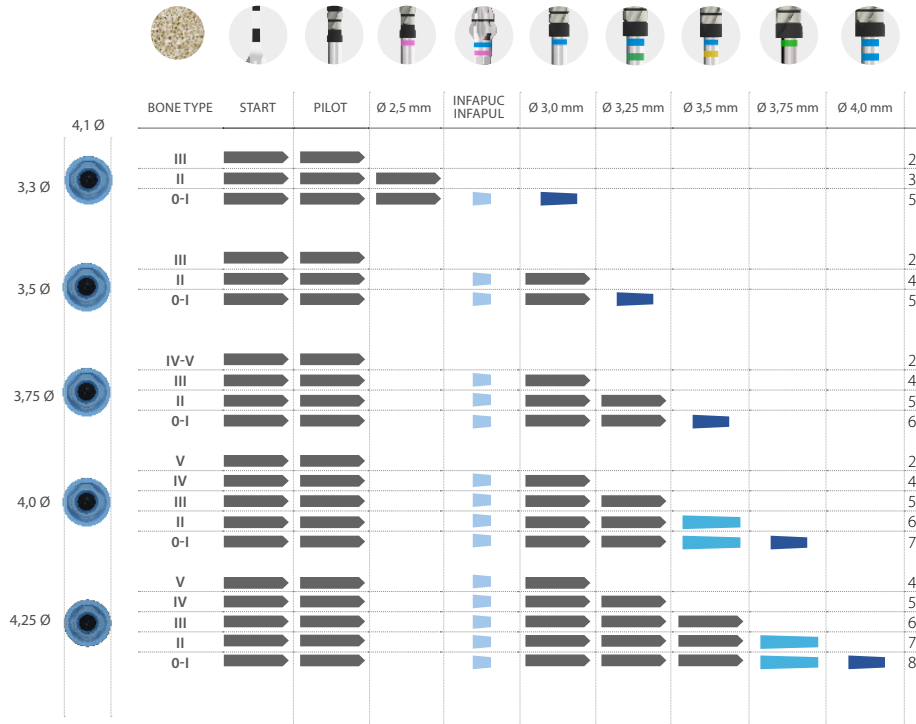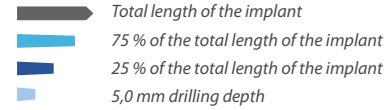

drilling
sequence guide

Implant lengths (from 5,5 to 8,5 mm)


-  Total length of the implant
-  Total length of the implant with specified diameter drill
-  5,0 mm drilling depth
-  2-3 mm drilling depth

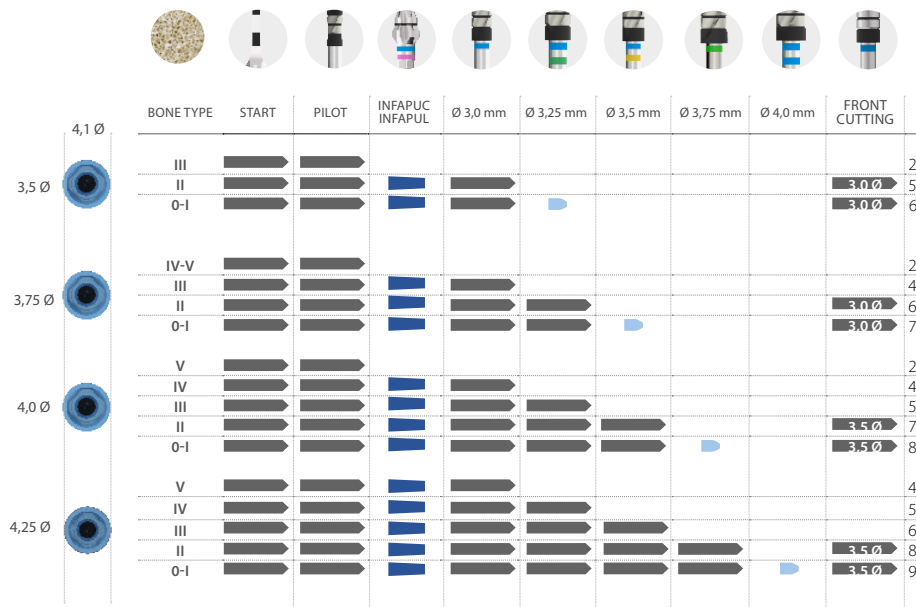

Implant lengths (from 8,5 to 15 mm)



 Total length of the implant
 6,5 mm drilling depth
 3 mm drilling depth

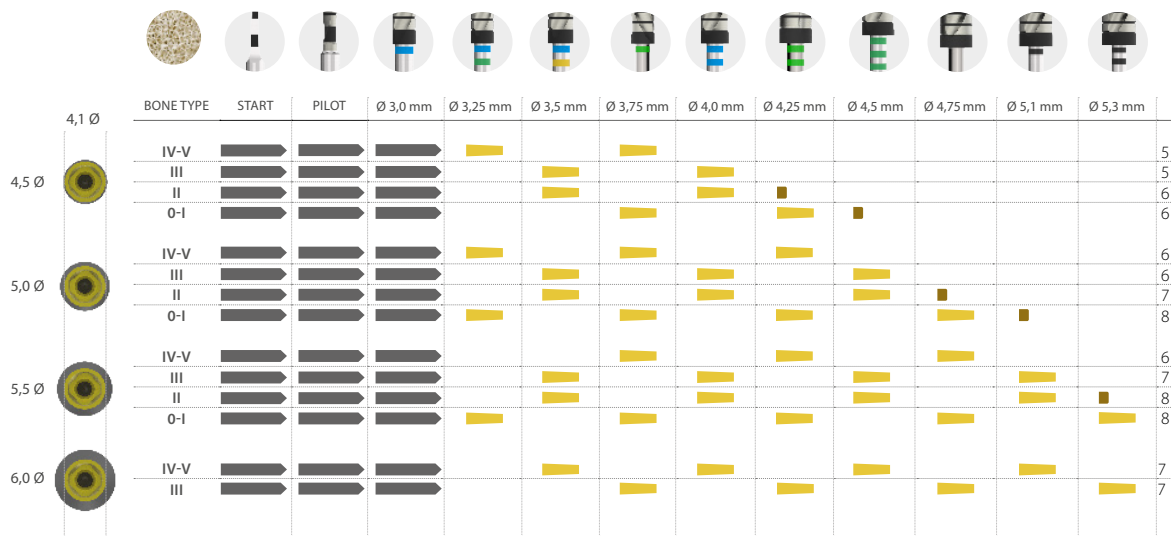




Implant lengths

(from 4,5 to 7,5 mm)

- Total length of the implant
- X,X Ø Total length of the implant with specified diameter drill
- 3 mm drilling depth

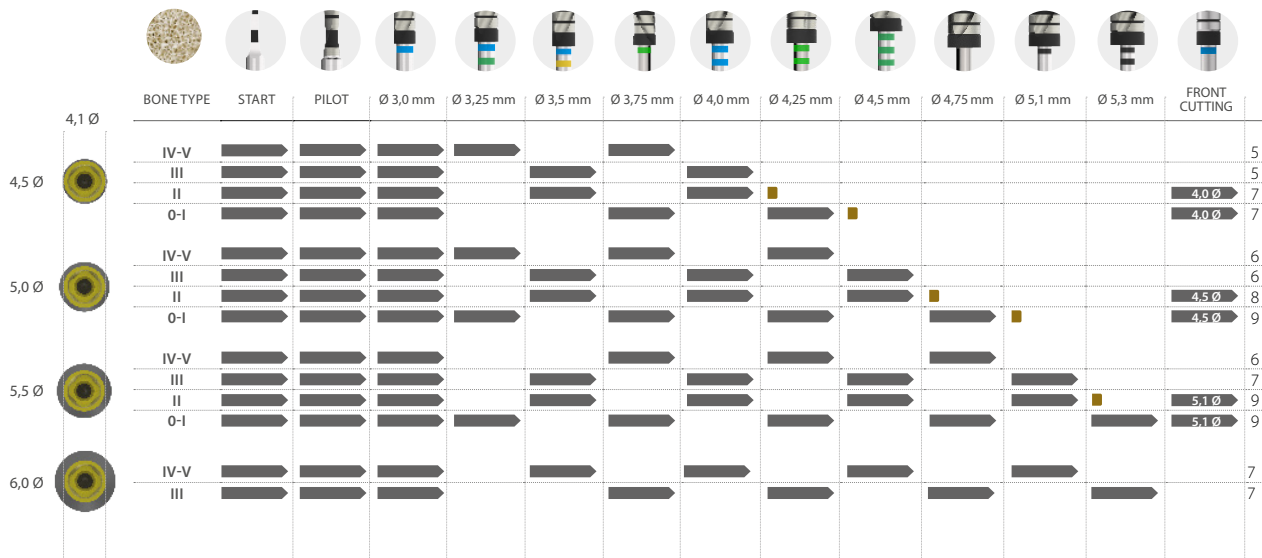

drilling
sequence guide