



Recommended drilling sequences

for BTI Externa implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

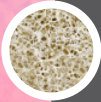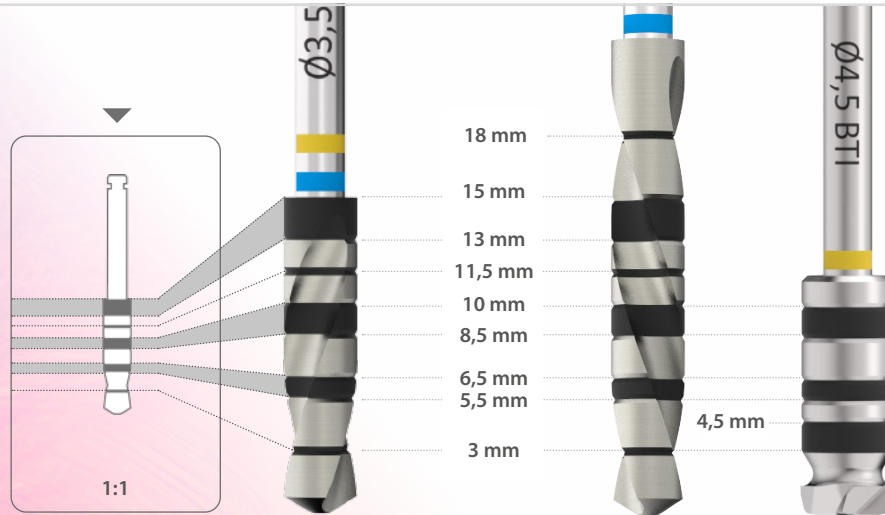

Class/Hounsfield

I 1050 - 1400

II 850 - 1000

III 550 - 800

IV 400 - 500

V 100 - 350

DRILL

Initial drill

Drill

Countersink drill

Front cutting drills

REVOLUTIONS

800 - 1.000 rpm

50 - 75 rpm

150 - 200 rpm

50 - 75 rpm

IRRIGATION

yes

no


no

no



Implant lengths

(from 10 to 15 mm)

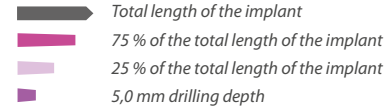




Implant lengths

(4,5 - 5,5 - 6,5 - 7,5 - 8,5 mm)



-  Total length of the implant
-  Total length of the implant with specified diameter drill
-  5,0 mm drilling depth (Do not use with 4,5mm length)

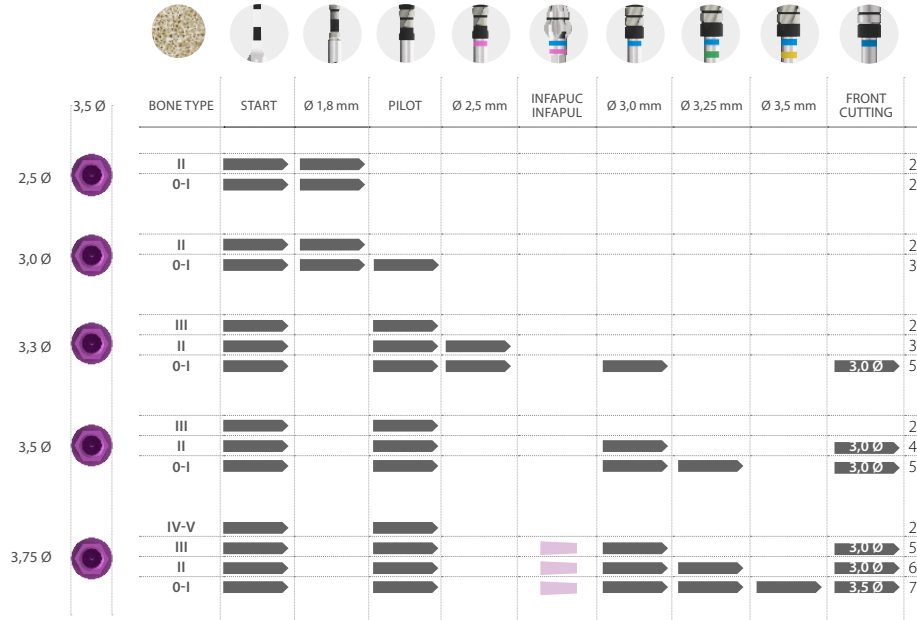



Implant lengths


(from 10 to 15 mm)

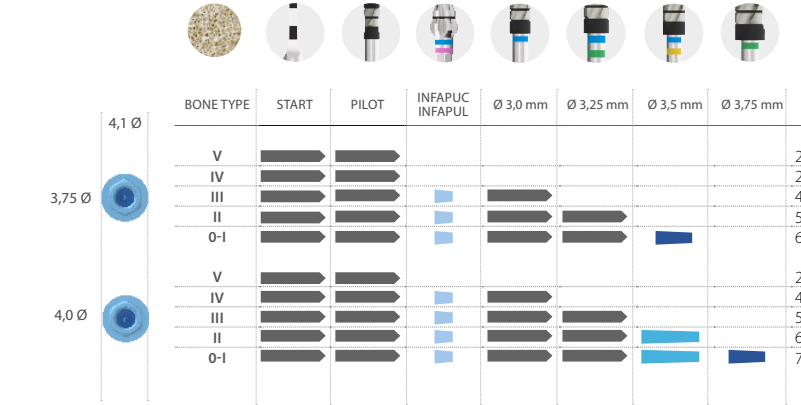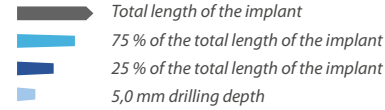



Implant lengths (6,5 - 7,5 - 8,5 mm)


- Total length of the implant
- 75 % of the total length of the implant
- 5,0 mm drilling depth
- 2-3 mm drilling depth

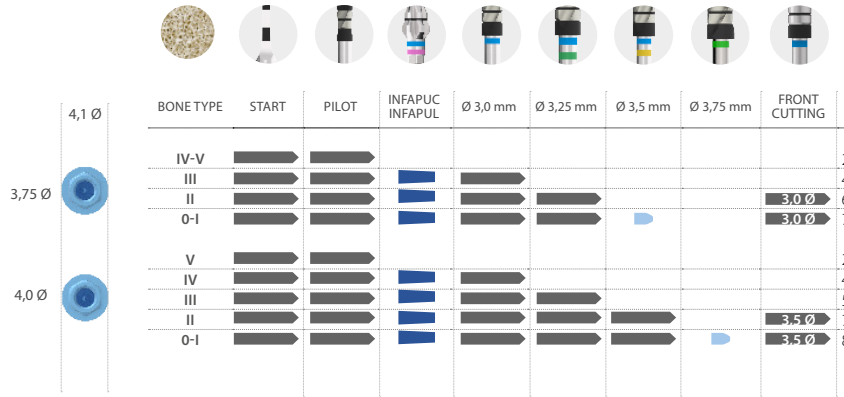

Implant lengths (from 10 to 15 mm)



-  Total length of the implant
-  75 % of the total length of the implant
-  25 % of the total length of the implant
-  4,5 mm drilling depth

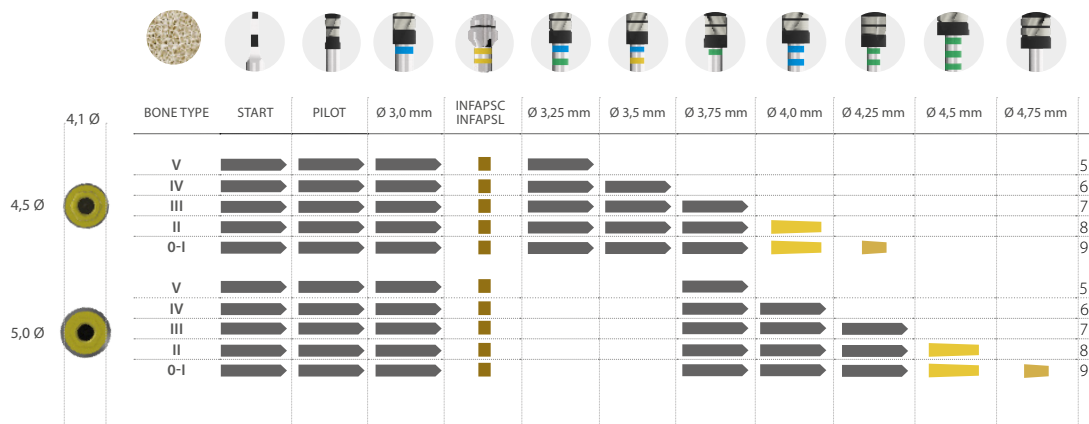

Implant lengths (6,5 - 7,5 - 8,5 mm)



 Total length of the implant
 Total length of the implant with specified diameter drill
 4,5 mm drilling depth

