


Recommended drilling sequences

for BTI Externa implants



NOTE: consult your distributor for the availability of the product in the different markets.

Drill depth marks

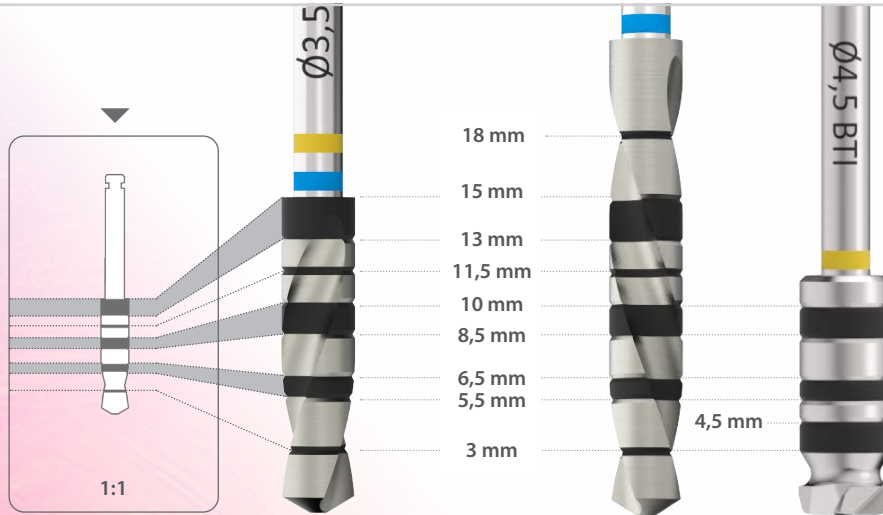

CLASS / HOUNSFIELD

0	1.400 - 1.200	III	800 - 500
I	1.200 - 1.000	IV	500 - 400
II	1.000 - 850	V	350 - 100

DRILL

Initial drill
Drill
Countersink drill
Front cutting drills

REVOLUTIONS

800 - 1.000 rpm
50 - 75 rpm
150 - 200 rpm
50 - 75 rpm


IRRIGATION

yes
no
no
no



Implant lengths

(from 10 to 15 mm)

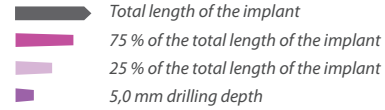




Implant lengths



(4,5 - 5,5 - 6,5 - 7,5 - 8,5 mm)

- Total length of the implant
- Total length of the implant with specified diameter drill
- 5,0 mm drilling depth (Do not use with 4,5mm length)

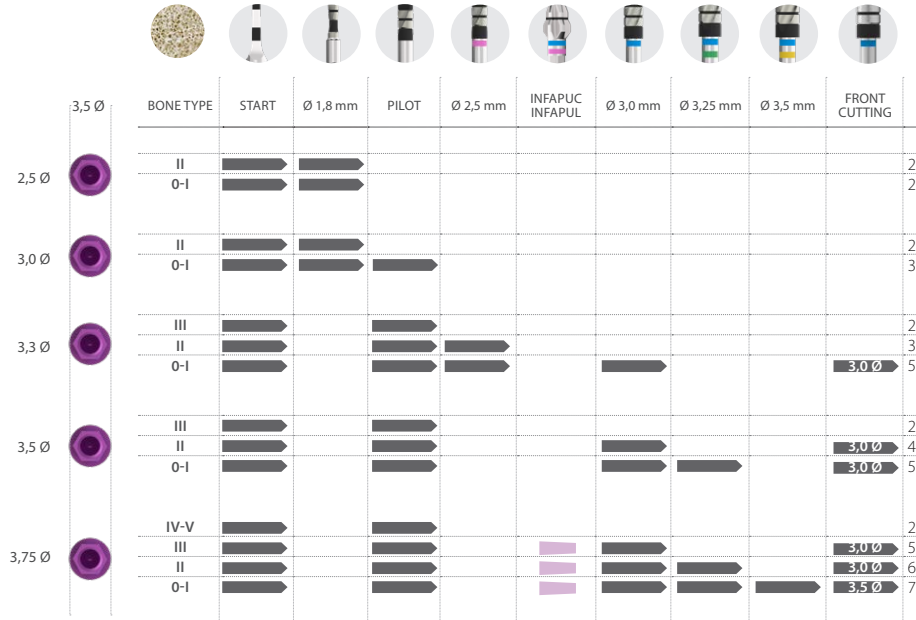




Implant lengths (from 10 to 15 mm)

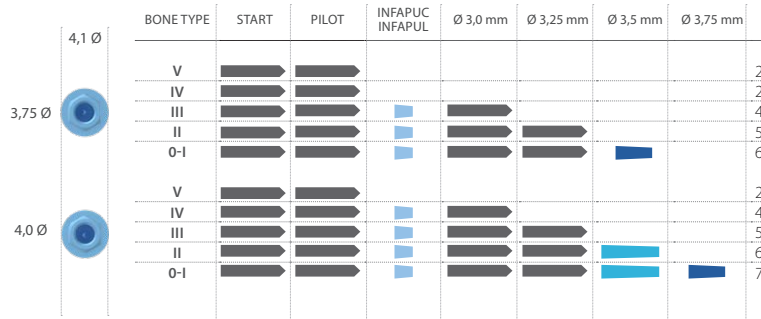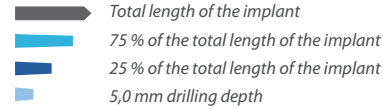



Implant lengths (6,5 - 7,5 - 8,5 mm)


- Total length of the implant
- 75 % of the total length of the implant
- 5,0 mm drilling depth
- 2-3 mm drilling depth

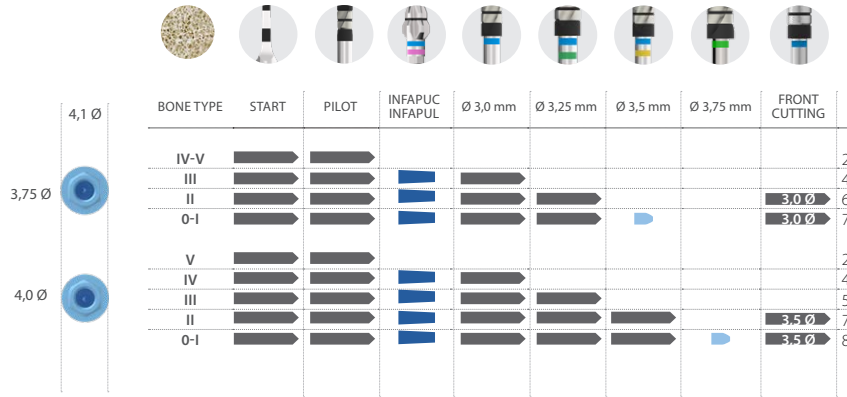

Implant lengths (from 10 to 15 mm)



- Total length of the implant
- 75 % of the total length of the implant
- 25 % of the total length of the implant
- 4,5 mm drilling depth

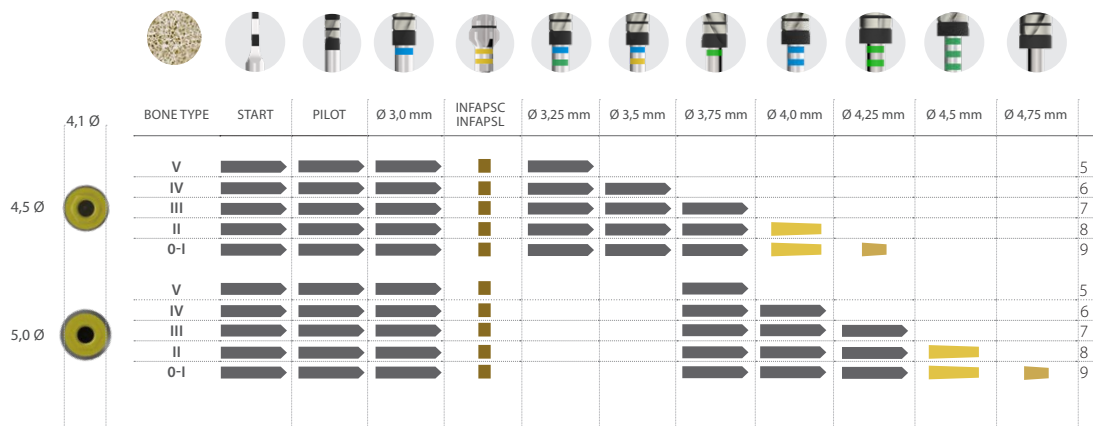

Implant lengths (6,5 - 7,5 - 8,5 mm)



 Total length of the implant
 Total length of the implant with specified diameter drill
 4,5 mm drilling depth

